



Handling And Wearing

To prevent the accidental ingestion of Faces, keep out of reach of small children. Never put Faces in the mouth for any reason, as this may cause a choking hazard.

Minor irritation and/or inflammation may occur as the ear becomes accustomed to the BTE instrument with a Face. If irritation persists, or allergic reaction occurs, discontinue use.

For Faces cleaning and storage, follow the instructions in the instrument's operation manual. If cracking or breakage occurs, discontinue use immediately and return to your hearing professional. Dispose of Faces in the proper waste or recycling manner.

Starkey Laboratories

P.O. Box 9457 • Mpls., MN USA 55440



Authorized European Contact:

Wm. F. Austin House, Bramhall Technology Park
Pepper Road, Hazel Grove, Stockport SK7 5BX,
United Kingdom



1. Attaching Faces:

Align the Face bottom with the instrument's battery compartment, swinging the Face over the instrument back until it snaps into place behind the earhook.



2. Removing Faces:

Insert a fingernail or small screwdriver between the Face and the instrument behind the earhook. Gently pull backwards until the Face disconnects from the instrument.