

Download the Thrive Hearing Control App for Android Devices

The Thrive Hearing Control app makes it easy to control and adjust your hearing aids with your compatible Android device. Downloading the app is easy; follow the step-by-step instructions below.

Step One: On a compatible Android device, go into the "Google play" store.



Step Two: In the search box at the top of the store, search for "Thrive Hearing Control."

A dark grey rectangular search bar with a back arrow on the left, the text "Thrive Hearing Control" in the center, and a magnifying glass icon on the right.

Step Three: Tap "FREE" to download the app to your device.

FREE

Step Four: Open the Thrive Hearing Control app by tapping on its icon.



Visit starkey.com/thrive-hearing for compatibility information.



Your **Thrive Hearing Control App User Guide** is a great resource for getting the best performance from your Livio AI hearing aids and Thrive app. Refer to your guide to find out more about these and other features:



- Integrated sensors and artificial intelligence features
- Automatic and custom memories
- Geotags
- SoundSpace
- Tinnitus management
- Find My Hearing Aids
- Alerts