

# Mini Remote Microphone



## Pairing

The Mini Remote Microphone must be paired to interact with compatible 2.4 GHz hearing aids on the Thrive platform. Pairing occurs once, then the hearing aids and Mini Remote Microphone will automatically recognize each other.

- 1 Power off your smart device or other accessories that were previously paired with your hearing aids.
- 2 Ensure the Mini Remote Microphone is powered on.
- 3 Power off, then power on your hearing aids to place in pairing mode.
- 4 Place your hearing aids within 6 inches of the Mini Remote Microphone.
- 5 Place the Mini Remote Microphone in pairing mode by double clicking the power button. The LED status indicator will alternately blink blue and red.
- 6 A green LED will blink for each hearing aid to confirm a successful pairing.



### Power On

Press the power button for 3 seconds and release when the LED illuminates.




### Power Off

Press the power button for 2 seconds and release when the LED turns off.




## How it Works

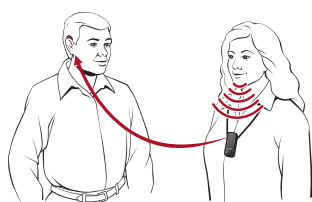

The Mini Remote Microphone focuses on a single speaker when background noise is present. The Mini Remote Microphone may also provide benefit for television watching.

Place the Mini Remote Microphone near the television speakers. To achieve a comfortable listening level, adjust the Mini Remote Microphone distance from the television speakers or modify the audio streaming volume level.

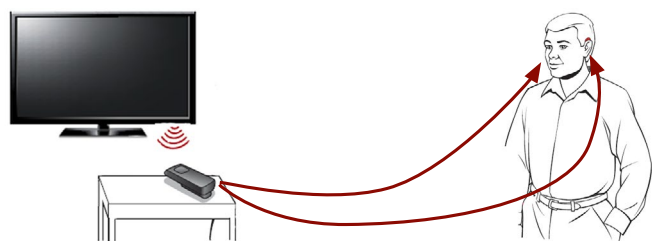

### To begin streaming from the Mini Remote Microphone:


- 1 Ensure the Mini Remote Microphone is powered on by pressing the power button for 3 seconds.
- 2 Streaming starts automatically in your hearing aids.
- 3 When finished, power off the Mini Remote Microphone to stop streaming.



## Use

---

- The Mini Remote Microphone can be clipped to clothing or worn on a lanyard.
- The Mini Remote Microphone operating range is up to 65 feet (20 meters).
- Place the Mini Remote Microphone close to the audio source for optimal results. The Mini Remote Microphone should be within 8 inches (20cm) of the speaker's mouth.
- The Mini Remote Microphone can stream for up to 10 hours on a full charge.

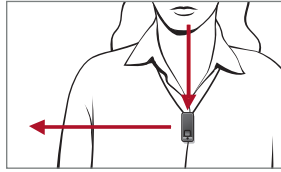

- The Mini Remote Microphone reaches a full charge in about 3 hours.
- When the Mini Remote Microphone battery is low, the LED status indicator will slowly pulse red and an audible indicator tone will play in your hearing aids every 5 minutes.
- The Mini Remote Microphone will momentarily mute if dropped and will resume streaming once the device is stabilized.
- Adjust the stream volume via the hearing aid user controls, the Remote Control accessory (if applicable) or in the Thrive Hearing Control app.